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Fax no. +49 (0)6261 9770-69


Sender:

Your order no. _____		Date _____	
Device type _____		Serial number _____	
<input checked="" type="checkbox"/>	Description	Item no.	Pieces
<input type="checkbox"/>	Display card	1.410.030	
<input type="checkbox"/>	Thermo controller	1.531.003	
<input type="checkbox"/>	Microswitch	6.561.001	
<input type="checkbox"/>	Power switch/filter	6.545.005	
<input type="checkbox"/>	Power cable 230V	6.593.013	
<input type="checkbox"/>	Power cable 115V	6.593.014	
<input type="checkbox"/>	Welding transformer 230V	6.531.023	
<input type="checkbox"/>	Welding transformer 115V	6.531.027	
<input type="checkbox"/>	Control card 230V	1.540.050	
<input type="checkbox"/>	Control card 115V	1.540.051	
<input type="checkbox"/>	Control card 100V	1.540.052	
<input type="checkbox"/>			
<input type="checkbox"/>			
<input type="checkbox"/>			
<input type="checkbox"/>			
<input type="checkbox"/>			

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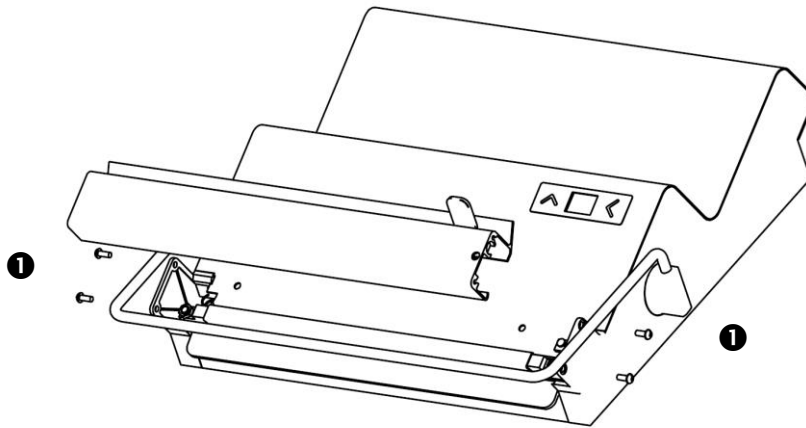1. Release securing screws ❶ on blade assembly and remove blade assembly


2. Release screws on guide carriage and remove guide carriage

Release screws on

Blade assembly
1.629.022

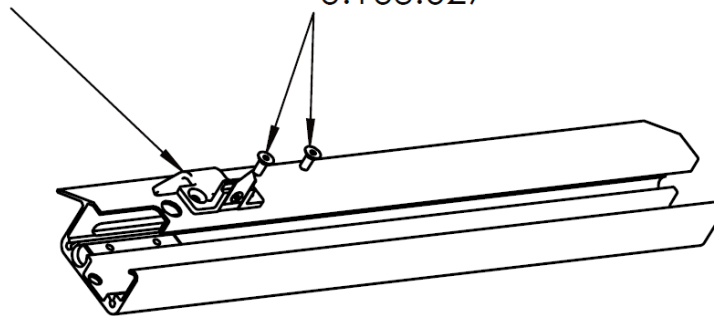Fixing screw
6.163.027

3. Insert new guide carriage and reinstall blade assembly in reverse sequence.

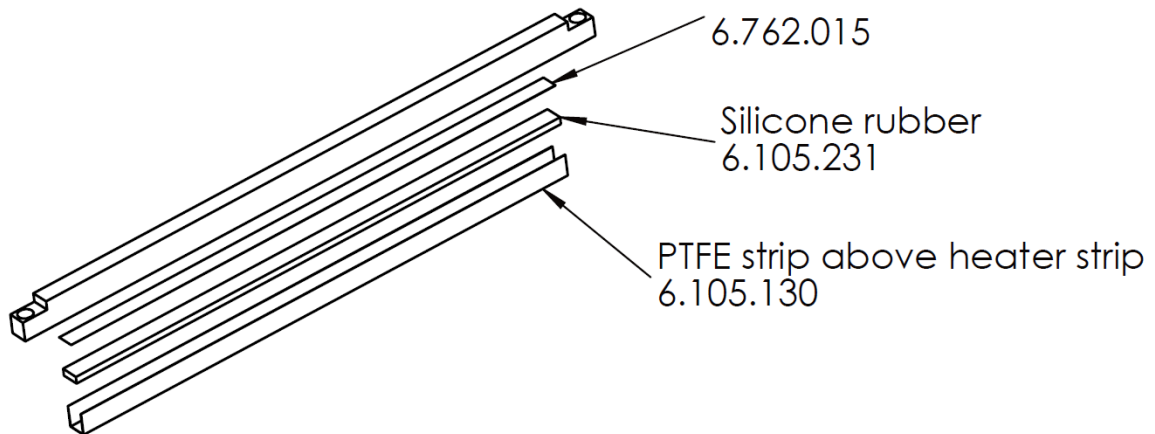
Important note !

The fixing screws ❶ must only be fixed with a torque wrench, torque maximal 1N.


4.6 Spare parts order – product number allocation

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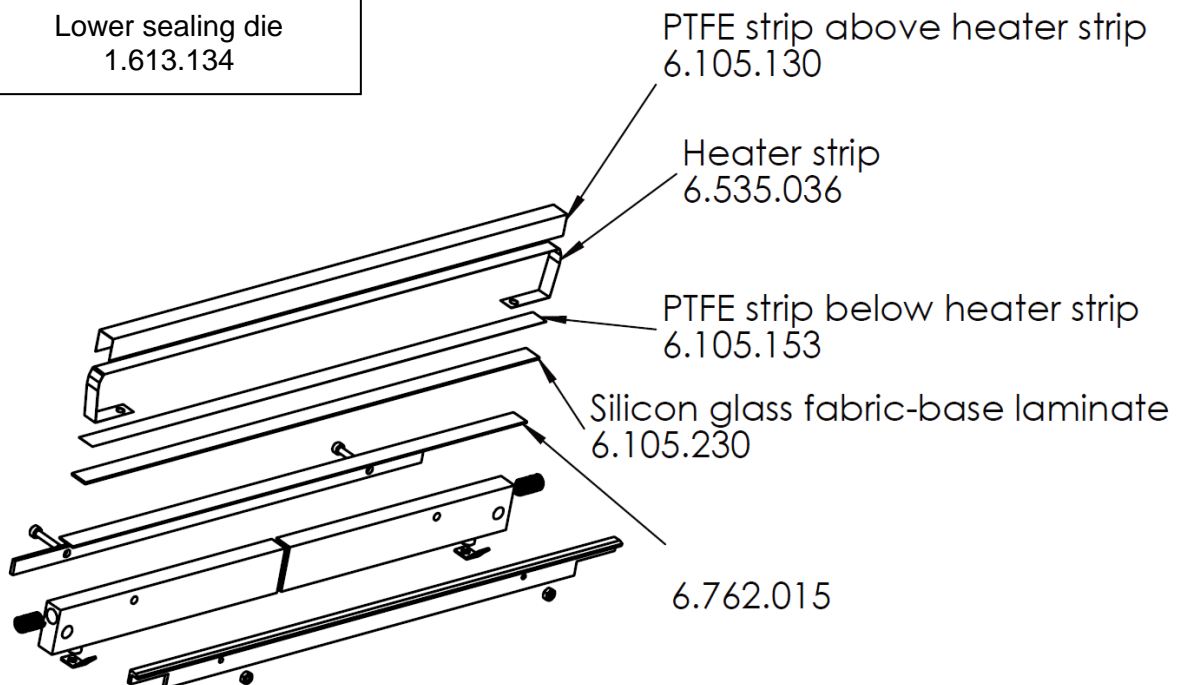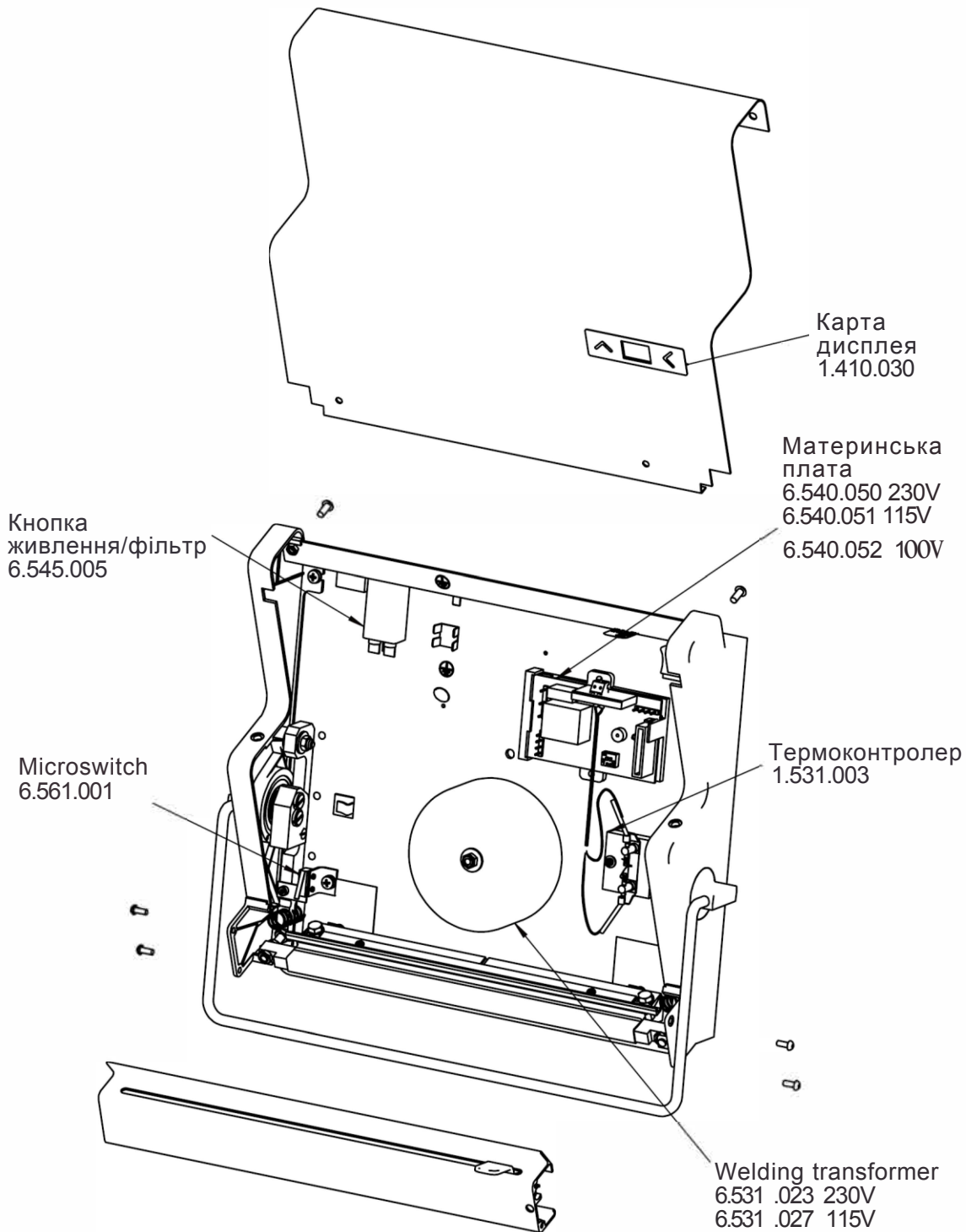
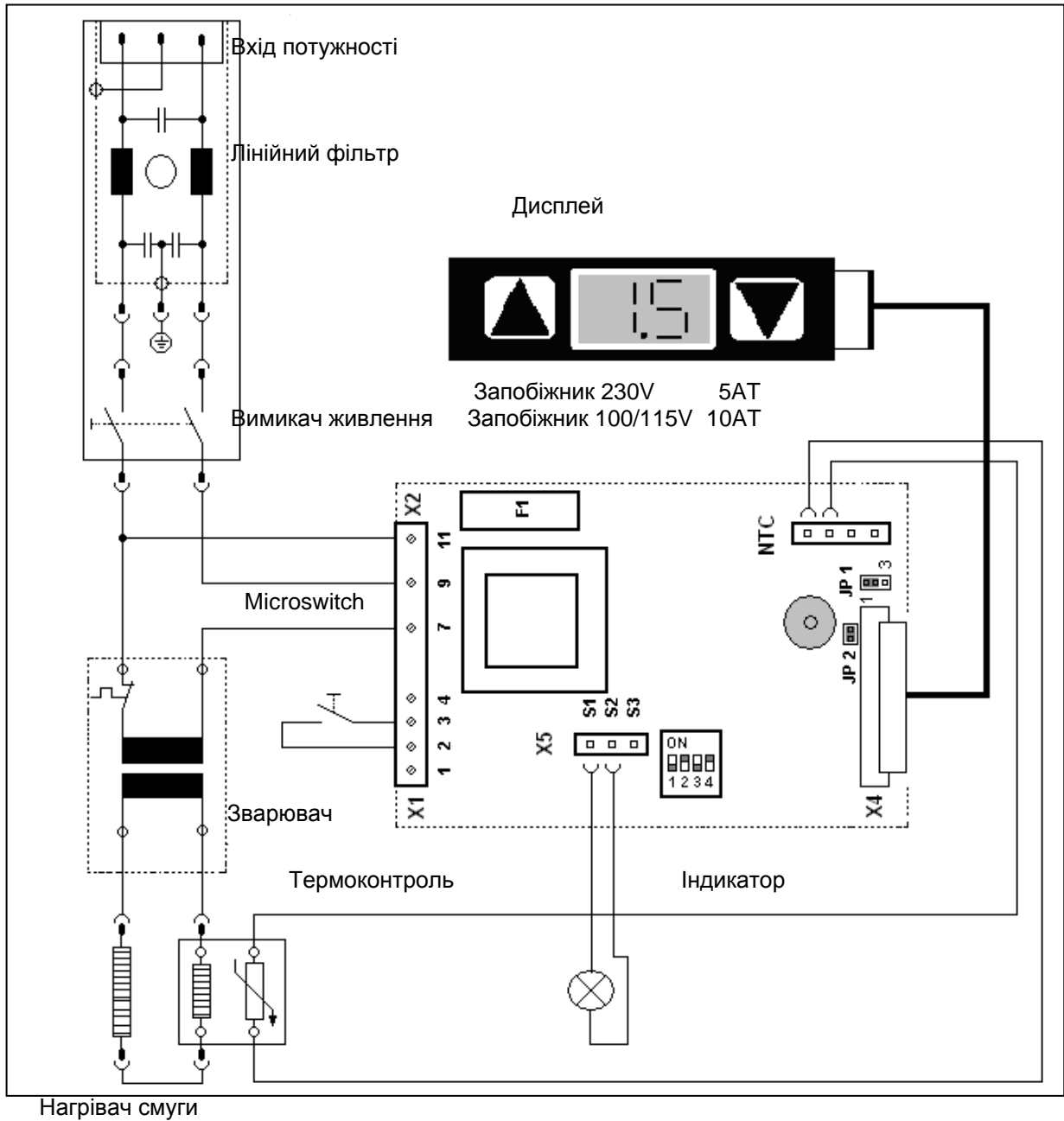
 hawo GmbH 74047 Obrigheim Germany	Konformitätserklärung - Declaration of Conformity Déclaration "CE" de Conformité Declaración de conformidad de la CE Dichiarazione di conformità - Declaração de conformidade	9.693.00BC																																																												
Gültig ab: 01.02.2015 Valid from:		Seite 1/1 Version 2.02																																																												
<p>Hiermit erklären wir, daß die Folienschweissmaschinen: Herewith we declare that the Foil sealing unit: Par la presente, nous déclarons que la gamme de Soudeuse de films plastique: Por la presente certificamos que las maquinas embolsadoras modelos: Dichiariamo con la presente che le macchine per saldatura di fogli: Por este meio se declara que as maquinas de selagem de folhas de plastico:</p> <p style="text-align: center;">hd 320 MS</p> <p>folgenden einschlagigen Bestimmungen und harmonisierten Normen entsprechen: complies with the requirements of the following regulations and harmonised standards: corresponde aux dispositions suivantes et standards harmonise: objeto de esta Declaración cumple con las siguientes disposiciones: Sono conformi alle seguenti disposizioni in materia nonché alle seguenti norme armonizzate: correspondem as seguintes determinações e normas harmonizadas:</p> <table border="0" style="width: 100%;"> <tr> <td style="width: 30%;">Niederspannungsrichtlinie</td> <td style="width: 30%;">Low voltage directive</td> <td style="width: 20%;">2014/35/EU</td> <td style="width: 20%;"></td> </tr> <tr> <td>Directive a basse tension</td> <td>Directiva de Baja Tension</td> <td></td> <td></td> </tr> <tr> <td>Direttiva sulla bassa tensione</td> <td>Directiva relativa a baixa - tensao</td> <td></td> <td></td> </tr> <tr> <td>EMV-Richtlinie</td> <td>EMC-directive</td> <td>2014/30/EU</td> <td></td> </tr> <tr> <td>Directive CEM</td> <td>Directiva da CEM</td> <td></td> <td></td> </tr> <tr> <td>Direttiva CEM</td> <td>Directiva CEM</td> <td></td> <td></td> </tr> <tr> <td>WEEE-Richtlinie</td> <td>WEEE-directive</td> <td>2012/19/EU</td> <td></td> </tr> <tr> <td>Directive WEEE</td> <td>Directiva de WEEE</td> <td></td> <td></td> </tr> <tr> <td>Direttiva WEEE</td> <td>Directiva WEEE</td> <td></td> <td></td> </tr> <tr> <td>RoHS-Richtlinie</td> <td>RoHS-directive</td> <td>2011/65/EG</td> <td></td> </tr> <tr> <td>Directive RoHS</td> <td>Directiva da RoHS</td> <td></td> <td></td> </tr> <tr> <td>Direttiva RoHS</td> <td>Directiva RoHS</td> <td></td> <td></td> </tr> <tr> <td>Harmonisierte Normen</td> <td>Harmonized standards</td> <td>EN ISO 12100/2010_07</td> <td>EN 60204-1/2007_06</td> </tr> <tr> <td>Standard harmonise</td> <td>Las normas armonizadas</td> <td>EN ISO 13857/2008_06</td> <td>EN 61000-6-1/2007_10</td> </tr> <tr> <td>Norme armonizzate</td> <td>Normas harmonizadas</td> <td></td> <td>EN 61000-6-3/2011_09</td> </tr> </table> <p>Verantwortliche Person for die Technischen Unterlagen, Name und Adresse siehe unten Responsible person for technical documentation see below La personne responsable pour la documentation technique est mentionnee au-dessous</p> <div style="text-align: center;">  <hr/> Torsten firhardt Prokurist / authorized officer </div> <p style="text-align: center;">hawo GmbH, Obere Au 2, D-74847 Obrigheim, Germany</p>			Niederspannungsrichtlinie	Low voltage directive	2014/35/EU		Directive a basse tension	Directiva de Baja Tension			Direttiva sulla bassa tensione	Directiva relativa a baixa - tensao			EMV-Richtlinie	EMC-directive	2014/30/EU		Directive CEM	Directiva da CEM			Direttiva CEM	Directiva CEM			WEEE-Richtlinie	WEEE-directive	2012/19/EU		Directive WEEE	Directiva de WEEE			Direttiva WEEE	Directiva WEEE			RoHS-Richtlinie	RoHS-directive	2011/65/EG		Directive RoHS	Directiva da RoHS			Direttiva RoHS	Directiva RoHS			Harmonisierte Normen	Harmonized standards	EN ISO 12100/2010_07	EN 60204-1/2007_06	Standard harmonise	Las normas armonizadas	EN ISO 13857/2008_06	EN 61000-6-1/2007_10	Norme armonizzate	Normas harmonizadas		EN 61000-6-3/2011_09
Niederspannungsrichtlinie	Low voltage directive	2014/35/EU																																																												
Directive a basse tension	Directiva de Baja Tension																																																													
Direttiva sulla bassa tensione	Directiva relativa a baixa - tensao																																																													
EMV-Richtlinie	EMC-directive	2014/30/EU																																																												
Directive CEM	Directiva da CEM																																																													
Direttiva CEM	Directiva CEM																																																													
WEEE-Richtlinie	WEEE-directive	2012/19/EU																																																												
Directive WEEE	Directiva de WEEE																																																													
Direttiva WEEE	Directiva WEEE																																																													
RoHS-Richtlinie	RoHS-directive	2011/65/EG																																																												
Directive RoHS	Directiva da RoHS																																																													
Direttiva RoHS	Directiva RoHS																																																													
Harmonisierte Normen	Harmonized standards	EN ISO 12100/2010_07	EN 60204-1/2007_06																																																											
Standard harmonise	Las normas armonizadas	EN ISO 13857/2008_06	EN 61000-6-1/2007_10																																																											
Norme armonizzate	Normas harmonizadas		EN 61000-6-3/2011_09																																																											
hawo GmbH Obere Au 24 74847 Obrigheim Germany	T+49 (0) 6261 19770-0 F+49 (0) 6261 19770-69 info@hawo.com www.hawo.com	Amtsgericht Mannheim: HRB 441011 Geschäftsführer: Hans Wolf und Christian Wolf Firmensitz: Obrigheim																																																												
This document and the contents thereof are considered proprietary and confidential information of hawo and disclosure to unauthorized individuals or dissemination, publication, or copying is prohibited without prior written consent by hawo GmbH, 74847 Obrigheim, Germany.																																																														